

Maintenance Kit for Diagnosing and
Treating Periodontal Disease


IM-Cure Kit

- Complete removal of Peri-implantitis
- Minimize damage to the surface of the fixture
- Easy access to Defect site




HROSSEN
IMPLANT

IM-Cure Kit Component

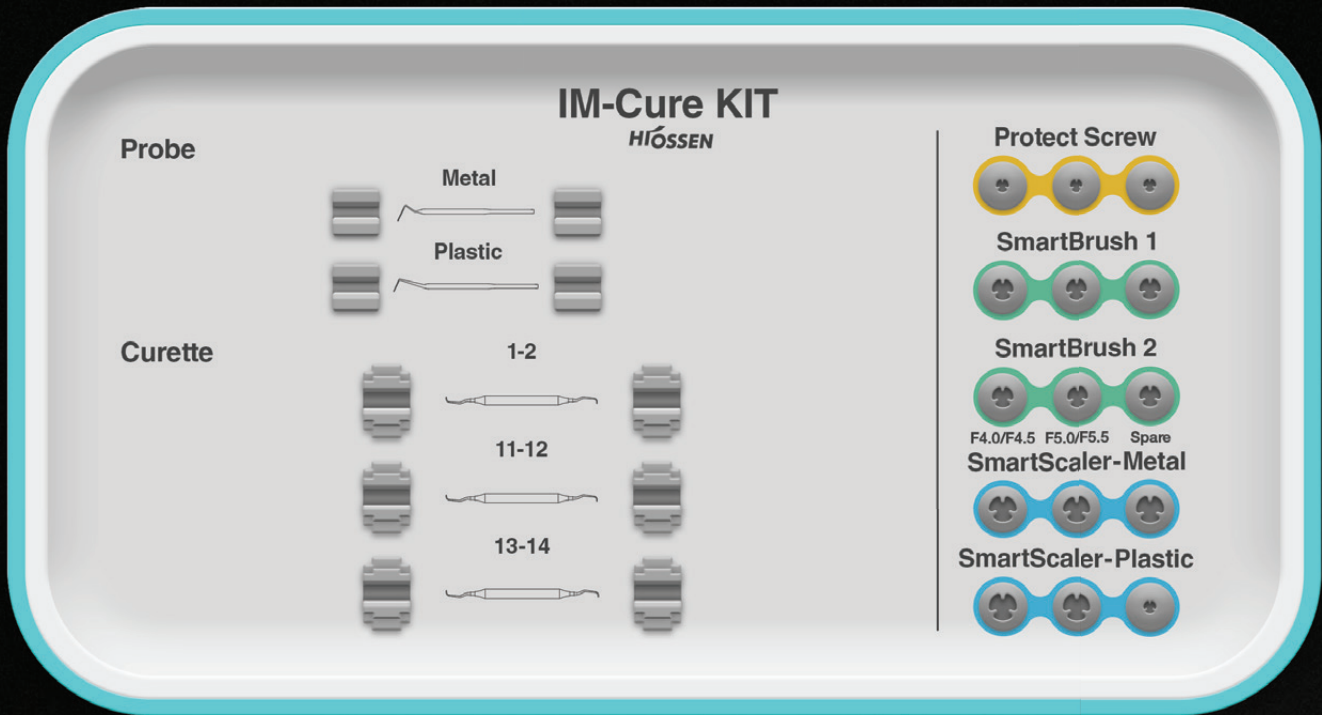


Probe (Metal/Plastic)

- Tool for probing gingiva and surroundings in order to diagnose periodontal diseases
 - **Metal** : diagnose peri-implantitis
 - **Plastic** : diagnose peri-mucositis
- 1mm increments of laser markings for accurate probing

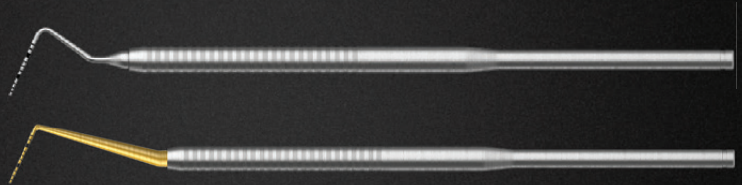


Curette

- Used for removing subgingival sediment that is attached to granulation tissue
- Used for removing plaque + subgingival sediment, and for performing root planning

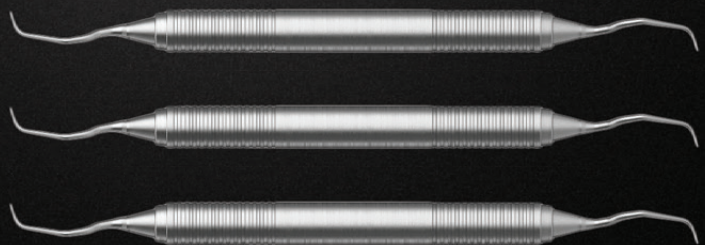


Protect Screw

- Prevents foreign substance from getting inside the fixture when using SmartBrush1 or SmartBrush2
- Use 1.2 Hex driver to deliver torque of 5Ncm to protect screw

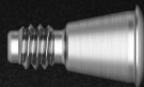

SmartBrush1

- Used for polishing off Peri-implantitis
- Used for cases of dehiscence and horizontal bone loss
- ※ Single use, Non-reusable

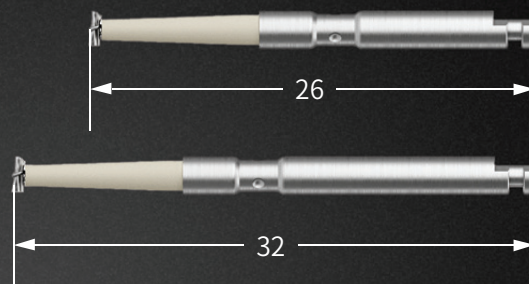

SmartBrush2

- Used for polishing off Peri-implantitis
- Used for cases of vertical bone loss
- ※ Single use, Non-reusable



Fine-Polishing

SmartScaler - Metal

- Fix to ultrasonic scaler in order to remove plaque or foreign substances from surface of fixture
- The tip is bendable for better accessibility




SmartScaler - Plastic

- Removes foreign substances from Abutment and crown
- ※ Prohibited from removing foreign substances on fixture surface



SmartScaler - Plastic


- Used to remove substances from abutment or crown by attaching the SmartScaler
- ※ Do not use to fixture surface

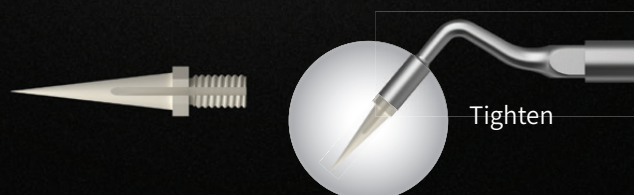

Eliminates Periodontitis with Multi-polishing Method

Step 1: Macro-Polishing

- Outstanding polishing ability by its design which fits the size of the Fixture threads
- Various brush types available for different cases

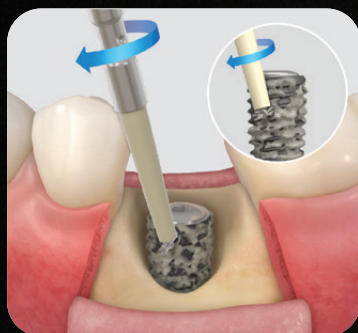

SmartBrush1

Used for dehiscence and horizontal bone loss



SmartBrush2

Used for vertical bone loss

Step 2: Fine-Polishing

- Improved accessibility due to its flexible tip
- Metal scaler minimizes fixture surface damage




SmartScaler - Metal


Can be freely bent in order to improve accessibility

Surgical Sequence


Removal of Peri-implantitis

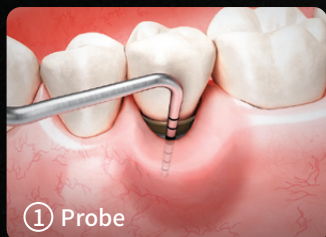


Diagnose Peri-implantitis with Probe

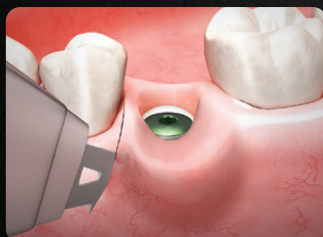

Remove prosthesis and make gingival incision

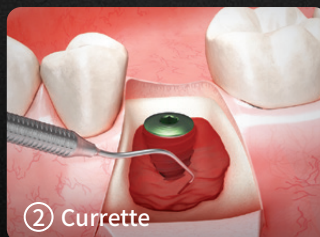


Remove granulation tissue with Curette

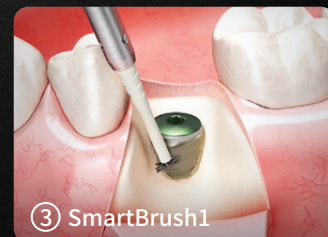

Perform Macro-Polishing with SmartBrush 1 and 2

Removal of Peri-Mucositis

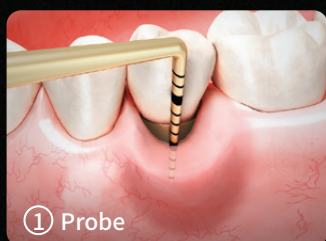

Diagnose Peri-Mucositis with a Probe



Conduct polishing of the prosthesis with the Plastic Scaler Tip



Perform fine-polishing with SmartScaler - Metal